WIN CARE



MANGAR Bathing Cushion

The Bathing Cushion is used to help someone who has difficulty getting in and out of the bath.

EASY TO FIT

- · Fitted and removed within seconds
- Some air can be left in the backrest for greater levels of comfort
- The design adapts to the size and shape of the bath, so no expensive bathroom alterations are required

SIMPLE TO MAINTAIN

- Lightweight and portable, it can be removed and cleaned with ease
- Rinse with clean water after each bath and leave it to dry inflated
- Compact and easy to store
- The material is designed to exceptionally high standards with extraordinary durability, strength & performance in the toughest of environments

SAFE TO USE

- Only lowers a person into the bath if there is enough battery power to raise them back out again
- Battery operated no electric cables
- Fully waterproof hand control (IP67)

IMPROVES WELL-BEING

- Perfect for a person who has difficulty getting in and out of the bath
- The cushion deflates allowing the user to lie flat in the bath, fully covered in water so benefiting from the therapeutic properties of bathing
- The user can take the bathing cushion away with them when travelling
- Can be removed from the bath when not in use so enabling a more flexible environment for shared bathrooms

TECHNICAL SPECIFICATIONS

Max user weight	150 kg
Max height	40 cm
Min height	Flat
Seat depth	44 cm
Width	58 cm
Total product weight	2 kg
Back/headrest length	33 cm
Airflo Compressor	12
Lifting performance per charge	Up to 8 lifts (4 bathing cycles)

Features

- **1. The air inlet valve can be rotated** allowing the hose to be led to the appropriate corner.
- 2. Pass the air hose through the loop in the corner of the backrest on the side that you propose getting in and out of the bath.
- 3. Made with a durable, anti-microbial material.
- **4. Strong suckers** ensure the bathing cushion doesn't move in the bath.
- 5. Safety feature which ensures there is enough charge to lift user out of bath.
- 6. Seams joined by high frequency welding for air tight construction

Accessories

Airflo 12 Compressor





Product code: MPCA030520



For more information visit www.mangarhealth.com Email: enquiries@winncare.uk Phone: 0800 2800 485



WIN CARE