Pedia[®] Soft

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UK

Information:

This pillow has been designed for the comfortable and relaxed placement of the head and neck of women during and after their pregnancy.

This pillow is made of a waterproof-breathable fabric envelope, soft to the touch and featuring a plantlike colour. The purpose of the membrane is to provide a good level of breathability of the pillow, but at the same time, it is also designed to act as a barrier, preventing the penetration of liquids under the cover. The cover is made of a fabric in which the portion that is in contact with the bodies of mothers and expecting mothers is classified as OEKO-TEX Standard class I, which meets the highest standard in terms of safety. This product is intended for both pre- and post-maternity usage, for both the support and the rest of the mother. Inside of the removable cover, you will find a cover that protects the pillow's so-called "memory foam", which features exceptional properties. This product adapts to the shape of its users and it was designed to meet current standards, including fire standards outlined in decree 2000-164, and it was designed in compliance with the EN 71 standard relating to seams. Item treated with a biocidal substance that is safe for the user. www.winnicare.fr

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Packaging and Storage:

The Pédia pillow comes in a travel bag that can be reused for travelling or to simply store your pillow in a dust-free environment; this bag is not a toy, it is your responsibility to keep it away from the reach of children.

This product should be stored in a location free of direct light and moisture.

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Weight: approx. 1.26 kg

Guarantee:

This product is guaranteed for 1 year

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UK

Information:

The cosy nest is recommended for all mothers and mothers-to-be by midwives.

The cosy nest has a waterproof/breathable fabric cover that is soft to the touch and in a natural colour. This membrane not only allows the product to breathe but also acts as a barrier to prevent liquid from seeping through the cover. The part of the fabric cover that comes into contact with the newborn's skin is classed as OEKO-TEX Standard 100 class I, which is the highest safety classification. Inside the removable cover you will find a cover containing small microbeads that can be moulded into shape as required. Suitable for resting babies between 0 to 5 months old with a maximum weight of 30 kg. Suitable only for short-term rest or naps.

This product has been designed to meet current standards, in particular the fire standards relating to decree 91-1292, and is designed to meet standard EN 71 relating to stitching. This product also meets safety requirements.

Item treated with a biocidal substance that is safe for the user. www.winnicare.fr

Instructions for Use:

Before placing your child on the product, it is important to make sure that the small pocket of microbeads inside the cover is well inside the specially designed kangaroo pocket and that you have spread out the beads by hand so that your child is stable and comfortable. Close the cover properly and place your child on his or her back on the green side of the cover. Make sure that you use the restraint system so that your child is safely positioned on the product. It is important to ensure that this restraint system is adjusted to fit your child so that he or she cannot slide around and is not too tightly restrained.

Care Instructions/Disinfection:

Green Outer Cover:

This cover is removable and can be machine washed at 95°. It can also be disinfected or dry cleaned. Before placing it in the washing machine, do not forget to remove the white pocket of beads from the kangaroo pocket, as this should not be machine washed. To prolong the life of the inner membrane, you are advised not to tumble dry this cover. Once the cover has been washed and dried, put the small pocket of beads back in the kangaroo pocket before replacing the cover. See assembly diagram for details.

White Inner Cover:

This contains the microbeads and it has been made so that the beads cannot be accessed or come out. The cover containing the beads should never be machine washed, tumble dried or ironed, but can be cleaned with approved surface detergents/disinfectants. These care recommendations apply to both the large pocket and the small pocket inside the kangaroo pocket of the cover.

Packaging and Storage:

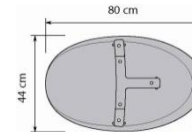
The cosy nest is supplied in a carry bag that can be re-used when you are on the move or simply used to protect your product from dust during storage. This bag is not a toy and it is your responsibility to keep it out of reach of children

Warning:

This item is not a toy. The cosy nest should never be used as a bassinet or bed and is not suitable for long periods of sleep. You are strongly advised not to place the cosy nest on a high surface. It should be used on a flat, horizontal and clear surface. It is the parents' responsibility to use it as specified in the instructions. This pillow should never be used without its green protective cover. Nor should the cosy nest be used to move or carry a child. There is a carry handle designed for moving the product but never with the child on it. It should never be used as a bassinet or car seat. If the product is accidentally punctured, it must not be used. To prevent this from happening, take care not to use sharp objects near the pillow. You should also keep the product away from any significant heat source or flame.

Characteristics/Description:

Dimensions: 80 cm long x 44 cm wide x 22 cm high
Weight: approx. 2.2 kg

Guarantee:

This product is guaranteed for 2 years.

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UK

Information:

This product is intended for pre- and post-maternity use allowing for proper posture to be retained during birth preparation exercises, maternal rest and breast-feeding. It is an easy-to-use accessory that provides valuable comfort and encourages good posture. It is recommended for all mothers and mothers-to-be by midwives.

This maternity and relaxation pillow has a waterproof/breathable fabric cover that is soft to the touch and in a natural colour. This membrane not only allows the product to breathe but also acts as a barrier to prevent liquid from seeping through the cover. The part of the fabric cover that comes into contact with the newborn's skin is classed as OEKO-TEX Standard 100 class I, which is the highest safety classification. Inside this removable cover you will find a cover containing small microbeads that can be moulded into shape as required. This product is designed to be used both antenatally and postnatally and allows the correct posture to be maintained during preparatory exercises for the birth, while the mother is resting and during breastfeeding.

This product has been designed to meet current standards, in particular the fire standards relating to decree 2000-164, and is designed to meet standard EN 71 relating to stitching.

Item treated with a biocidal substance that is safe for the user. www.winncare.fr

Instructions for Use:

Lying down: on your side: for example, in the final months of your pregnancy, lay a leg or an arm on the pillow when resting. This position is comfortable and relaxing for your spine and limbs.

For breastfeeding: this pillow is perfect for breastfeeding. It supports and relieves your arms, shoulders and neck. Its flexible, comfortable padding also ensures that your baby is well supported. You are reminded never to leave a child unattended.

For relaxation: in addition to its uses in pregnancy and breastfeeding, this pillow can be used in a number of ways to assist with relaxation. For example, to relax your lumbar region or the muscles in your neck and shoulders.

Care Instructions/Disinfection:

Green Outer Cover :

This cover is removable and can be machine washed at 95°. It can also be disinfected or dry cleaned. However, to prolong the life of the inner membrane, you are advised not to tumble dry this cover.

White Inner Cover :

This contains the microbeads and it has been made so that the beads cannot be accessed or come out. The cover containing the beads should never be machine washed, tumble dried or ironed, but can be cleaned with approved surface detergents/disinfectants.

Packaging and Storage:

The maternity pillow is supplied in a carry bag that can be re-used when you are on the move or simply used to protect your product from dust during storage. This bag is not a toy and it is your responsibility to keep it out of reach of children.

Warning:

This item is not a toy. The maternity pillow should never be used as a bassinet or bed and should not be used on a high surface. It is the parents' responsibility to use it as specified in the instructions. The pillow should never be used without its protective cover. Nor should the pillow be used to move or carry a child. If the product is accidentally punctured, it must not be used. To prevent this from happening, take care not to use sharp objects near the pillow.

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UK

Information:

This Half Moon pillow will accompany you throughout your pregnancy and after the birth of your child. It features an ergonomic design that allows it to absorb the warmth of the neck and to perfectly adapt to its shape for optimal support. This pillow was conceived with an ergonomic design that perfectly adapts to the shape of your neck for optimal comfort. This product is made of a waterproof-breathable fabric envelope that is soft to the touch and features a plantlike colour. The purpose of the membrane is to provide a good level of breathability of the pillow, but at the same time, it is also designed to act as a barrier, preventing the penetration of liquids under the cover. The cover is made of a fabric in which the portion that is in contact with the bodies of mothers and expecting mothers is classified as OEKO-TEX Standard class I, which meets the highest standard in terms of safety. This product is intended for both pre- and post-maternity usage, for both the support and the rest of the mother. Inside of the removable cover, you will find a cover that protects the pillow's so-called "memory foam", which features exceptional properties. This product adapts to the shape of its users and it was designed to meet current standards, including fire standards outlined in decree 2000-164, and it was designed in compliance with the EN 71 standard relating to seams. Item treated with a biocidal substance that is safe for the user. www.winnicare.fr

Instructions for Use:

Remove the cushion from its reusable packaging and follow the diagram below for proper use of your pillow.

Care Instructions/Disinfection:

Green Outer Cover:

This cover is removable and allows for machine washing at a temperature of 95°. It can be disinfected or washed dry. To favour the inner membrane's durability, it is recommended that you not dry this cover by machine. Refer to the assembly instructions.

White Inner Cover:

This cover is present in order to provide the viscoelastic foam with additional protection. This sewn cover must never be washed or dried by machine.

Packaging and Storage:

The Half Moon Pillow comes in a plastic bag; this bag is not a toy, it is your responsibility to keep it away from the reach of children. This product should be stored in a location free of direct light and moisture.

Warning:

This pillow is in no way intended to be used without its protective green cover. Avoid humid contact with the foam, which could alter its properties. Do not machine wash the foam. Make sure that you keep the product away from all sources of extreme heat and flames.

Characteristics/Description:

Dimensions: 28 cm long x 24 cm wide x 10 cm high
Weight: approx. 0.34 kg

Guarantee:

This product is guaranteed for 1 year.

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